

LLLT Series

Wearable neck laser therapy device

650nm

The laser U-shaped shoulder and neck massager is convenient and easy to use! Its gentle red light massage alleviates pain. Using laser diodes as the output medium, it delivers energy directly to the discomfort area more effectively than other methods. With interchangeable continuous and pulse modes, just 30 minutes of use promotes circulation and relieves discomfort. Use it daily as part of your wellness routine!



Feature

Promotes Circulation, Warm Massage

Developed to relieve soreness

- Relieve discomfort
- Alleviates fatigue
- Promotes blood circulation

I Specification

Wavelength	650nm
Laser specifications	19pcs
Output power	Class 2M, <1mW
Battery capacity	Lithium polymer battery 1900mAh
Product size	L190/ W160/ H53 ±5mm
Weight	348 ±10g
Operating environment	Temperature 5~40 degrees; Humidity 0~80%
Transportation and storage environment	Temperature 5~40 degrees; Humidity 0~80%

【 Cautions 】

- Avoid light exposure to your eyes directly for protect your eyes.
- Transformer is prohibited for use which may cause dangerous event.
- Do not use fast charging for this product, as it may cause danger.
- Please keep clean on the area of laser output, prevent the dirties affect energy transmission when use.
- Please keep it away from the touch by children or pets in order to avoid eating or touch components accidentally.
- Do not use on pregnant women, who with skin disease, malignant tumor, wear pacemaker, initial wound within 72 hours, physical abnormalities and abnormal sensory nerves. Avoid to use it during the skin is sensitive, fragile and in rash or wounds condition. Please stop using it immediately and consult doctor if you feel any discomfort.
- Don' t use it with electronic medical equipment in order to avoid the malfunction or dangerous accidents.
- Please store it in dry, cool place and away from water. Please pay attention on moisture and falling during use.

I LECC TECHNOLOGY CO., LTD.

2F No.2, Ziqiang 3rd Rd., Zhongli Dist., Taoyuan City, 320023, Taiwan.
 TEL | 03 4511967 VAT Number | 16708380
 FAX | 03 4511923 Email | sales@lecc.com.tw

I SHANGHAI LECC OPTO CO., LTD.

No.1939, Da Ye Rd., Feng-Xian Dist., Shanghai City, 201402, China.
 TEL | 021 57407400 TAX Number | 913100007630197034
 FAX | 021 57407403 Email | sales@lecc.com.tw